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Mosaic™ nurse call unit

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Llamadas de enfermeras Mosaic™

Call from patient's room (with call confirmation)

- The patient calls by pressing the bed-head pushbutton cord in their room
- The door unit indicator and the corridor bulkhead light slowly flash red
- In the nurses' station, the room number is displayed, the indicator slowly flashes red and the audible signal beeps slowly
- The door unit indicators and the corridor bulkhead lights in all the rooms where a nurse is present change from steady yellow to flashing yellow
- From the nurses' station, the nurse acknowledges the call by pressing the mute button. The red indicator changes from flashing to steady and the audible signal stops
- The door unit indicator and the corridor bulkhead light change from flashing red to steady red

Call from patient's bathroom

- The patient calls by pulling the cord in the bathroom
- The bathroom and door unit indicators and the corridor bulkhead light rapidly flash red
- In the nurses' station, the room number is displayed, the indicator rapidly flashes red and the audible signal beeps rapidly
- The door unit indicators and the corridor bulkhead lights in all the rooms where a nurse is present change from steady yellow to rapidly flashing yellow

Since this is a priority call, the nurse is unable to press the mute button and must go to the room.

Nurse present

- When entering the room, the nurse presses the door unit pushbutton
- The red door unit indicator and the corridor bulkhead light go out. The door unit indicator and the corridor bulkhead light return to steady yellow
- In the nurses' station, the red indicator goes out and the yellow indicator returns to steady state

1st example : Cancelling the call

- When leaving the room, the nurse presses the door unit pushbutton and clears the call. All the indicators and the bulkhead light go out

2nd example : Nurse's call for assistance

- In the room, the nurse activates the emergency call by pressing the bed-head cord pushbutton
- The door unit indicators and the corridor bulkhead lights rapidly flash red and yellow alternately
- In the nurses' station, the indicator rapidly flashes red and yellow alternately. The audible signal sounds intermittently
- In the other rooms where a nurse is present, the door unit indicators and the corridor bulkhead lights rapidly flash yellow
- When entering the room, the second nurse presses the door unit pushbutton
- When leaving the room, the nurse presses the door unit pushbutton and clears the call. All the indicators and the bulkhead light go out

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